

# TREATMENT OF SKIN LESIONS





**THIS BOOKLET IS SHOWCASING PATIENTS  
BEFORE AND AFTER TREATMENT WITH  
CRYOSUGERY.**

**EACH PATIENT TREATED IS A TESTAMENT TO  
OUR CONTINUED FOCUS ON SERVING  
PROFESSIONALS AND PATIENTS WITH THE  
BEST CRYOSUGERY.**



# **TREATMENT OF SKIN LESION IN A HORSE**



# CASE

**Specie:** Horse

**Age:** 11 years

**Breed:** Araber

**Name:** Joana

**Gender:** Mare

**Indication:** Sarcoid, placed at left udder. Sarcoid is the most common form of skin cancer in horses.


**Treatment** (under aneesthesia):

Freeze/thaw times for 1. treatment: 5 x 15 seconds

Freeze/thaw times for 2. treatment: 6 x 15 seconds

Freeze/thaw times for 3. treatment: 5 x 15 seconds

Freeze/thaw times for 4. treatment: 5 x 15 seconds

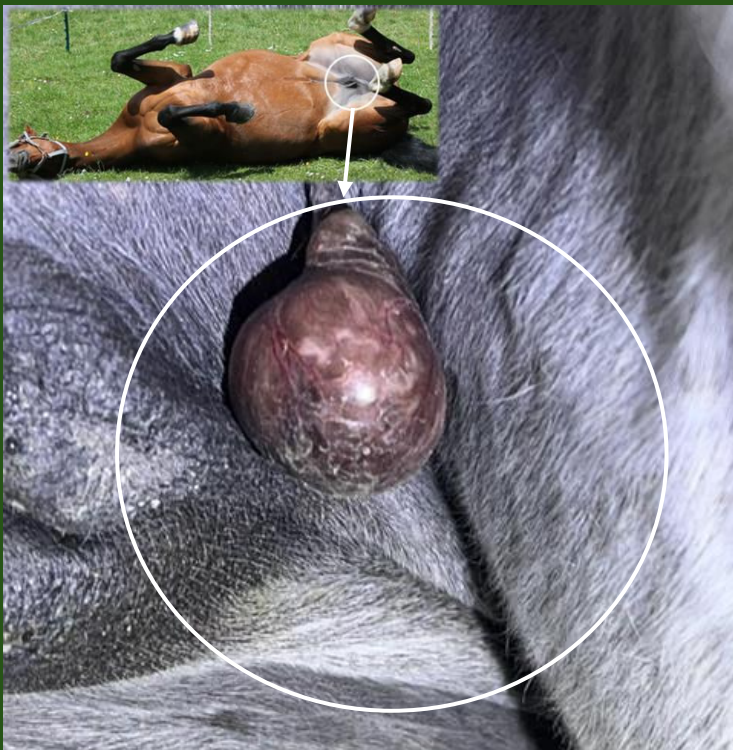

# Treatment Process

The horse Joana – SARCOID

- 1. Treatment: 5 x 15 seconds
- 2. Treatment: 6 x 15 seconds

## 1. TREATMENT: 5 x 15 seconds



## 2. TREATMENT AFTER 7 DAYS: 6 x 15 seconds



# Treatment Process

The horse Joana – SARCOID

- 3. Treatment: 5 x 15 seconds.
- 4. Treatment: 5 x 15 seconds.

**THE SARCOID FELL OFF 14 DAYS AFTER THE 1. TREATMENT**


## **3. TREATMENT AFTER 7 DAYS: 5 x 15 seconds** **About to fall off.**



## **4. TREATMENT AFTER 7 DAYS: 5 x 15 seconds** **Result: The sarcoid fell off.**



The horse owner remarked, "I was pleasantly surprised by how quickly the treatment was completed and how minimal the procedure turned out to be."

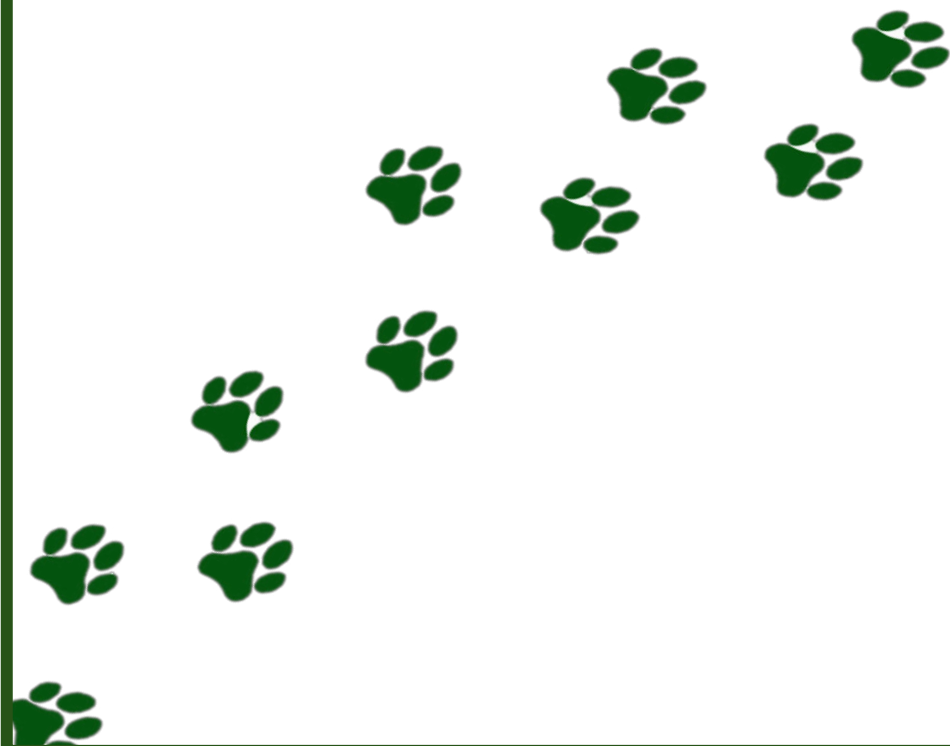

# TREATMENT OF SKIN LESIONS IN DOGS



# Treatment Process

## The dog Mollie – WART

- Sprayed once 5 x 15 seconds
- No side effects were observed.
- Wart fell of 3 days after treatment

**Before**



**Before close-up**



**After**



**After - close-up**



# Treatment Process


## The dog Alba – WART

- Sprayed 5 x 5 seconds
- No side effects were observed
- Wart fell off 3 days after treatment

**Before**



**Before close-up**

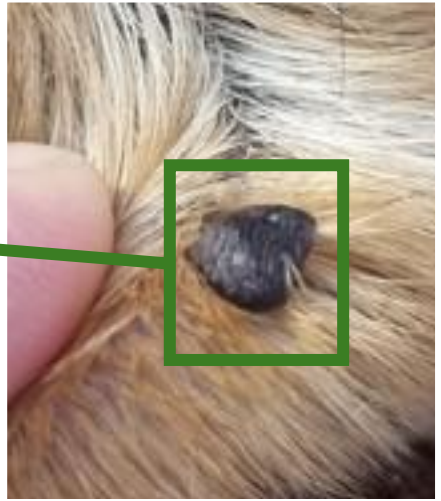

**After**



**After - close-up**



# Treatment Process

## The dog Eddie – WART

- Sprayed once 5 x 15 seconds
- No side effects were observed
- Wart fell of 2-3 days after treatment

**Before**




**Before close-up**



**After**



**After - close-up**

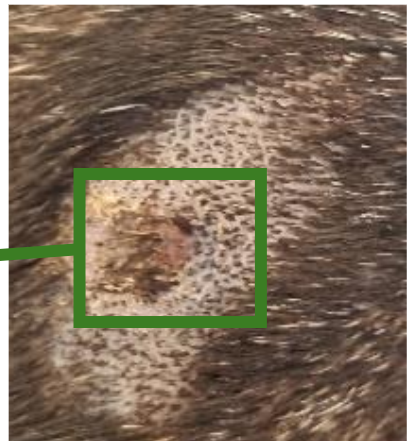

# Treatment Process

## The dog Wito – WART

- Sprayed once 5 x 15 seconds
- No side effects were observed
- Wart fell of 2 days after treatment

**Before**



**Before close-up**



**After**



**After - close-up**



# Treatment Process

## The dog Ellie – PAPILLOMA

- Sprayed once 5 x 15 seconds
- No side effects were observed
- Wart fell off 3 days after treatment

**Before**



**Before close-up**



**After**



**After - close-up**



# Treatment Process

## The dog Bella – WART

- Sprayed once 2 x 3 seconds
- No side effects were observed
- Wart fell of 2-3 days after treatment

**Before**




**Before close-up**



**After**



**After - close-up**

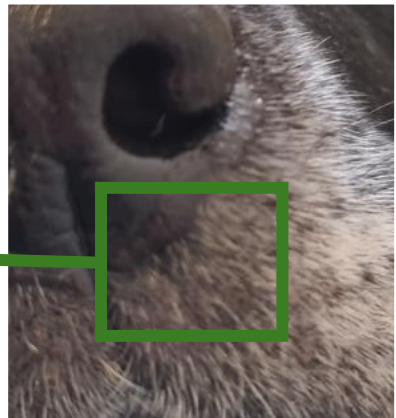

# Treatment Process

## The dog Antonia – WART

- Freeze/thaw time 3 x 5 seconds
  - Cryo used as anesthesia
- Surgical removal following anesthesia
  - No side effects observed

**Before**



**Before close-up**



**After**



**After - close-up**




# Treatment Process

## The dog Benno – WART in the perianal region

- Freeze/thaw time 3 x 5 seconds
  - Cryo used as anesthesia
- Surgical removal following anesthesia
  - No side effects observed

**Before**

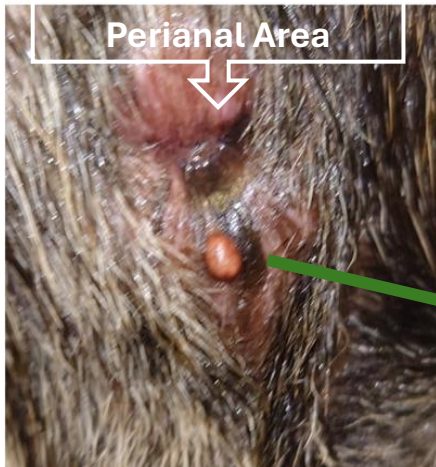

**Before close-up**



**After**



**After - close-up**





# TREATMENT OF SKIN LESION IN CATS



# Treatment Process


## The cat Ludvig – SQUAMOUS CELL CARCINOMA

- Sprayed once 1 x 45 second
- No side effects were observed
- Successful treatment outcome observed after 1 month

**Before**




**Before close-up**

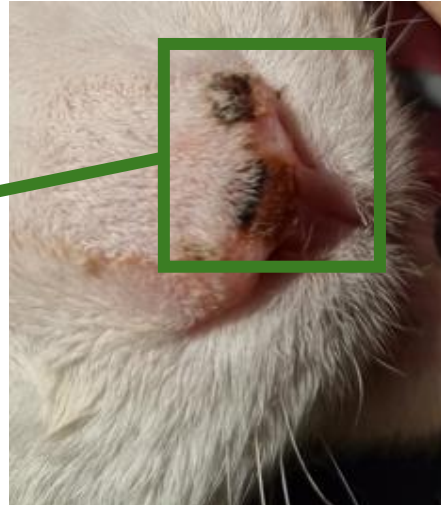

**After**



**After - close-up**

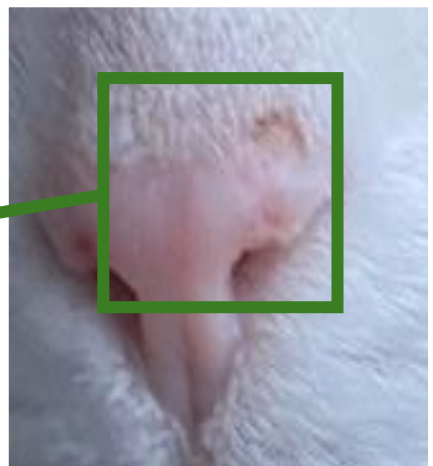

**The veterinarian confirmed Ludvig shows no signs of cancer**



**BIBAWO Medical A/S is a Danish family-owned medical device company that is ISO 13.485 certified.**

**We have developed & introduced cryosurgery in several European countries, the USA and India. We have a continued focus on developing and introducing the best available cryosurgical equipment to the benefit of professionals and patients worldwide – in doing so we honor our values of being “Decent, Long-Term Focused and Business Oriented”.**

